



WishBone Medical Broken Screw Removal System

Instructions for Use

WishBone Medical Broken Screw Removal System



Manufactured by:

WishBone Medical, Inc.
100 Capital Drive
Warsaw, IN 46582

P: +1 (574) 306-4006

F: +1 (574) 566-1600

	<ul style="list-style-type: none"> Caution: Federal law restricts this device to sale by or on the order of a physician.
5.1.1	<ul style="list-style-type: none"> Manufacturer
5.1.3	<ul style="list-style-type: none"> Date of Manufacture
5.4.2	<ul style="list-style-type: none"> Single-use
5.1.6	<ul style="list-style-type: none"> Catalog number
5.1.5	<ul style="list-style-type: none"> Lot number
5.1.5	<ul style="list-style-type: none"> Quantity
5.4.4	<ul style="list-style-type: none"> Caution: See instructions for use
5.2.4	<ul style="list-style-type: none"> Sterile – Irradiation
5.1.4	<ul style="list-style-type: none"> Use by date
5.2.6	<ul style="list-style-type: none"> Do not re-sterilize
5.2.6	<ul style="list-style-type: none"> Not made with Natural Rubber Latex
5.2.8	<ul style="list-style-type: none"> Do Not Use If Package is Damaged
5.2.7	<ul style="list-style-type: none"> Non-Sterile

Symbols: ISO 15223-1

CONTENTS

The package contains one or several surgical instrument(s) for use with the removal of broken or damaged screws.

DESCRIPTION

The following instruments are included within the contents of the kit: Trephine, EZ Out, Conical Extractor, T-Handle, and Inline Handle.

MATERIAL

The instruments are made from medical grade stainless steel & plastic.

INTENDED USE

The WishBone Broken Screw Removal System includes screwdriver shafts for all WishBone Medical screw sizes and drives, and extraction instruments for extracting broken and/or damaged screws. The instruments are provided in kits grouped by screw size.

ADVERSE REACTIONS

Possible reactions may include but are not limited to:

- Clinical failure due to inappropriate usage.
- Necrosis due to thermal load (power driven tools).

SAFETY PRECAUTIONS

- Prior to use, thoroughly read these instructions for use.
- Keep these instructions for use accessible to all staff.
- The use of a surgical instrument for tasks other than those for which they are intended may result in damaged or broken instruments or patient injury.

STERILE:

Caution: This is a single-use device. Re-use of this device can result in the transfer of materials not limited to bone, tissue, blood or infectious disease. This device has been sterilized with gamma irradiation. Do not re-sterilize. Do not use if package is open or damaged.

HOW SUPPLIED/STORAGE:

The instruments are packed in protective packaging that is labeled to its contents. All instruments are supplied sterile.

- Always store the instruments in the original protective packaging.
- Store the instruments in a dry and dust-free place (standard hospital environment).

INSPECTION:

Before use, inspect the instrument box carefully. Do not use when sterile barrier is visibly damaged.

FOR FURTHER INFORMATION

Please contact WishBone Medical Inc. or your authorized representative if further information about this product is needed.

© WishBone Medical, Inc.
February 2022
IFU-BSK Rev E