

BROKEN Screw


REMOVAL SYSTEM

THE SINGLE-USE, STERILE PACKED SOLUTION
EVERY OR NEEDS FOR BROKEN & STRIPPED SCREWS.

- Three kits supporting screw removal of different sizes (2.7mm, 3.5mm and 4.5mm)
- EZ Out for extraction of stripped or damaged screw heads
- Trepine and Conical Extractor for removal of screw shafts
- T-Handle and In-line Handle offered for intra-operative flexibility
- Pristine and ready for use at a moments notice!



BROKEN SCREW REMOVAL SYSTEMS

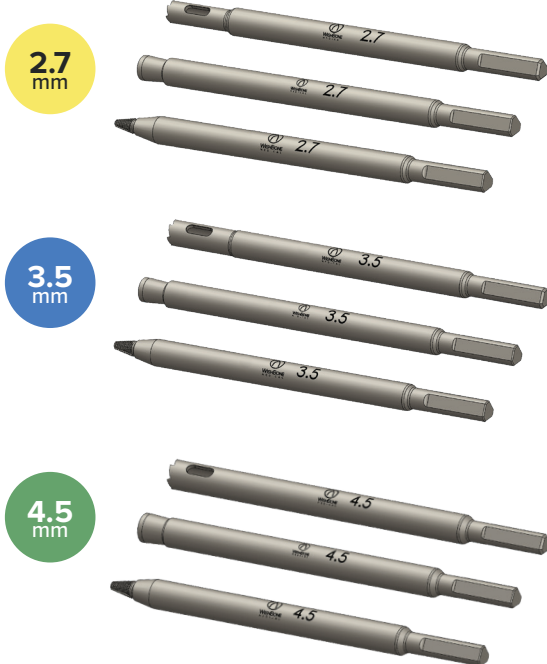

STERILE KIT #	DESCRIPTION
KIT-BSK-270	2.7mm Broken Screw Removal System

STERILE KIT #	DESCRIPTION
KIT-BSK-350	3.5mm Broken Screw Removal System



STERILE KIT #	DESCRIPTION
KIT-BSK-450	4.5mm Broken Screw Removal System

BROKEN Screw

REMOVAL SYSTEM

In-line or T-Handle can be used with EZ Out, Trepine, or Conical Extractor based on spatial clearance and/or surgeon's preference.

STRIPPED OR DAMAGED
SCREW HEAD REMOVAL
WITH EZ OUT



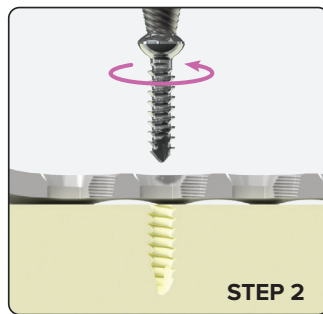
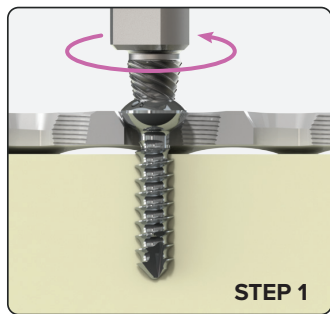


Step 1: The EZ Out is placed into the stripped or damaged screw head.

Step 2: The handle is rotated in a counter-clockwise direction until the screw is removed.

 = Reverse/Counterclockwise

SCREW SHAFT
REMOVAL

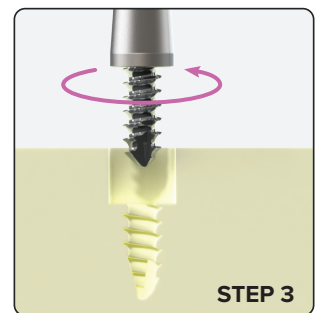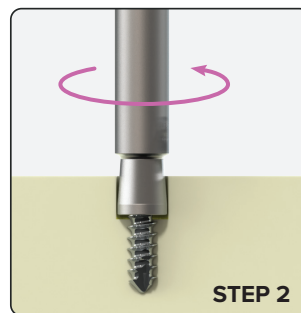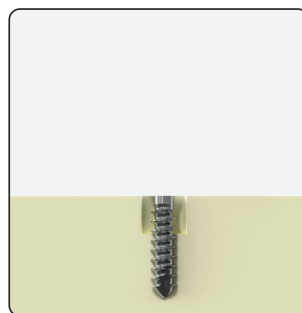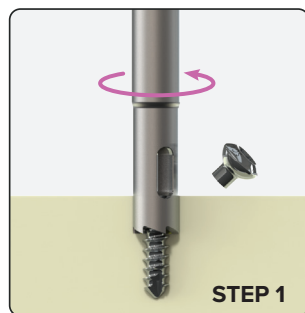

Step 1: The Trepine is placed over the shaft of the screw and rotated in a counter-clockwise direction until the Trepine bottoms out on the shaft of the screw or until the counter-bore is deep enough for the Conical Extractor to be used.





Step 2: The Conical Extractor is placed into the counter-bore with the shaft of the screw within the opening of the Conical Extractor.

Step 3: The handle is rotated in a counter-clockwise direction until the screw is removed.

Note: If experiencing difficulty engaging the threads of the screw shaft with the Conical Extractor, consider off axis alignment for engagement.

Caution: Federal law restricts this device to sale by or on the order of a physician.



 100 Capital Drive
Warsaw, IN 46582
 +1-574-306-4006
 WishBoneMedical.com


We **WORK** so they can **PLAY**.™

For ordering information, please contact your local rep or visit the WishBone Medical product catalog page online at www.wishbonemedical.com/catalog